

NEODENT

Orthodontic Anchorage

PRODUCT FEATURES:

- Available in Titanium alloy as per ASTM-F136 (V);
- Self-perforating;
- Collar height;
 - Low: 0 mm;
 - Medium: 1 mm.
- Hole diameter: 0.7 mm;
- Hex diameter: 2,7mm

Indications:

- Implants for orthodontic movement.

Drilling features:

- Drilling speed: 200 rpm;
- Placement speed: 30 rpm;
- Torque resistance of up to 10 N.cm (Ø 1.3 mm) and 20 N.cm (Ø 1.6 mm).



Orthodontic Anchorage Implant Package.



Remove the cap to access the implant.



Implant capture with Orthodontic Anchorage Contra-Angle Connection.



Implant placement with Contra-Angle Connections (105.039 or 105.040).



Option of manual implant insertion using a Handle Anchorage Implant Driver (104.033) or Torque Wrench Adaptor for Contra-Angle Connections (105.025).



Implant placed.

	Low Collar				Medium Collar			
	5 mm	7 mm	9 mm	11 mm	5 mm	7 mm	9 mm	11 mm
Ø 1.3								
	109.484 109.485 109.486				109.487 109.488 109.489			
Ø 1.6								
	109.701 109.493 109.494 109.495				109.702 109.496 109.497 109.498			

► Instruments



Drills for Orthodontic Anchorage

- :: Available in stainless steel;
- :: Recommended for type I and II bones;
- :: Marks refer to Implant length (5, 7, 9 and 11mm)

	Ø 1.1 mm	Ø 1.3 mm	Ø 1.6 mm
Straight Piece	103.042	103.078	103.043
Contra-Angle	103.044	103.079	103.045



Orthodontic Anchorage Adaptor Connections

- :: Connections for placing Anchorage Implants with Torque Wrench and Contra-Angle;
- :: Torque Wrench Adaptor Contra-Angle Connections (105.025).

Short	Long	Wrench
105.040	105.039	105.025

Orthodontic Anchorage Implant Driver

- :: Available in stainless steel;
- :: Orthodontic Anchorage Implant manual placement.

104.033

